

CYMED

life without limitations

MICROSKIN[®] based
OSTOMY POUCHING SYSTEMS

*Product
Catalog*
2013-2014

Ben Moon
cancer survivor
surfer
photographer
ostomate

800 582-0707
www.cymedostomy.com

Cymed Ostomy Co.



TABLE OF CONTENTS

●	What makes a MicroSkin [®] Ostomy Pouch better?	3
●	Urostomy Pouching Systems	4
●	Colostomy One-Piece Pouching Systems	6
●	Colostomy Two-Piece Pouching Systems	10
●	Ileostomy One-Piece Pouching Systems	12
●	Ileostomy Two-Piece Pouching Systems	14
●	Accessories	15
●	Pediatric Pouching Systems	16
●	MicroSkin [®] Hints and Tips	18
●	Applying a MicroSkin [®] Pouch	19



What makes a MicroSkin® Ostomy Pouch better?

The MicroSkin® Adhesive Barrier

The MicroSkin® Adhesive Barrier is what keeps your pouch attached to your body. MicroSkin® is made of a latex-free, hypo-allergenic, polyurethane polymer film. Designed with an emphasis on comfort and durability, at just $\frac{1}{1000}$ of an inch thick and the ability to stretch up to 500%, the MicroSkin® barrier accomplishes a comfort unparalleled with any other pouching system on the market.



MicroSkin® already has a well-established reputation of strength and durability, both in the water and under extreme situations, which has made it a favorite among active ostomates; yet the comfort and dependability can be enjoyed by any ostomate. MicroSkin® offers innovative design benefits that far exceed any other pouching system on the market today!

- **BREATHABLE:** air and perspiration pass through the MicroSkin®, but fluid and bacteria cannot. This allows your skin to heal and maintain itself naturally, helping prevent against skin breakdown and irritation.
- **WATERPROOF:** since fluids can't enter MicroSkin®, security can be maintained while bathing or swimming.
- **THIN:** extremely Low-Profile, measuring just $\frac{1}{1000}$ of an inch, MicroSkin® comfortably contours and stretches with your body's natural curves and movements.
- **DEPENDABLE:** proven under the most demanding conditions: John Dermengian (Ileostomate) wore the same MicroSkin® pouch as he swam 2.4 miles, rode his bike 112 miles, and then ran 26.2 miles, as he competed in the 2003 Ironman Triathlon in Hawaii.



MicroSkin® is barely visible when in place around your stoma.

It stretches and moves with you.

The Cymed Hydrocolloid Washer

Washers provide extra protection and support to the skin surrounding the stoma without sacrificing comfort. Naturally extracted from citrus peels, Cymed's washers are a specially formulated pectin-based hydrocolloid. Washers can provide some absorption and longer wear time for wounds, as well as providing some minor convexity for recessed stomas.

- MicroSkin® barriers are available with integrated washers, or can be purchased separately.
- Sizes: available in **Cut-to-Fit** or in any of our **Pre-cut** sizes.
- Thickness: available as either **Thin** (1mm) or **Thick** (2mm).

Thin Washer

use for
Colostomy
Urostomy

Thick Washer

use for
Ileostomy
Colostomy with
liquid output
Urostomy

Cymed Pouches are Medicare reimbursable.

Call 800 582-0707 for more product information

Urostomy Pouching Systems

9" Clear Urostomy Pouch

Box of 10
HCPCS A4391

- MicroSkin® Adhesive Barrier
- Anti-Reflux Design
- Includes Two Night Drainage Connectors
- Snap Cap Closure

Plain
Barrier—
No Washer



86300*

Thin
Washer



86300W

Thick
Washer



86300E

Cut-to-fit for Stomas
up to 1 3/4" (45mm)

Pre-cut sizes:

For 3/4" (19mm) stoma	86319	—	—
For 7/8" (22mm) stoma	86322	86322W	86322E
For 1" (25mm) stoma	86325	86325W	86325E
For 1 1/8" (29mm) stoma	86329	86329W	86329E
For 1 1/4" (32mm) stoma	86332	86332W	86332E
For 1 3/8" (35mm) stoma	—	86335W	86335E
For 1 1/2" (38mm) stoma	—	86338W	86338E

*86300 series is cut 1/8" larger than stoma size to ensure that the transparent film does not push on the stoma in any way. Cut-to-fit for Stomas up to 1 3/4" (45mm)

life without limitations

Frank Luciano was diagnosed with bladder cancer in 2005. His wife Janet was a great support system for him for the first year. Now Frank has it down and can change his urostomy pouch in 2 minutes!



Frank's stoma is flush with his skin—and he has a hernia. The MicroSkin® urostomy pouch #86300 is ideal for this situation. This pouch stays in place for Frank even with a daily swim, golf, and tending his favorite hibiscus.



Urostomy Pouching Systems

11" Large-Size Clear Urostomy Pouch with Large Plain Barrier

- Large-Size MicroSkin[®] Adhesive Barrier
- Anti-Reflux Design
- Includes Two Night Drainage Connectors
- Snap Cap Closure

Cut-to-fit for Stomas up to 2 ½" (64 mm)

* Large Barrier washer now available separately. (See pg. 15 for more details)

Box of 10
HCPCS A4391



86800X

Cymed Night Drain Adaptor

- EZ grip molded design
- For use with urostomy pouching system

Two standard night drainage connectors are included in every box of urostomy pouches.

3 tubes/package
HCPCS A9900



87000

Urostomy pouch is attached to the body with MicroSkin[®]



No belts or plastic means the MicroSkin[®] pouch is not discernible even in snug-fitting jeans.

Useful Tips for Urostomy Users

- **Do Cymed's urostomy pouches work with a flush or recessed stoma?**
Yes, the urine simply flows over the skin-like MicroSkin and there is no need for convexity. The MicroSkin[®] Plain Barrier or the barrier with the thin washer are appropriate for a flushed or recessed stoma.
- **How does Cymed's urostomy pouch connect to overnight drainage?**
Our night drainage connector is a vinyl surgical tube that pushes on to the end of the pouch and to the overnight drainage bag. A molded adaptor with a wider diameter providing more leverage for application is also available (87000).

Colostomy One-Piece Pouching Systems

One-Piece 11" Drainable Pouch

Box of 10

HCPCS A4388

- MicroSkin® Adhesive Barrier
- Thin washer

CLEAR



81300

OPAQUE



81400

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	81322	81422
For 1" (25mm) stoma	81325	81425
For 1 $\frac{1}{8}$ " (29mm) stoma	81329	81429
For 1 $\frac{1}{4}$ " (32mm) stoma	81332	81432
For 1 $\frac{3}{8}$ " (35mm) stoma	81335	81435
For 1 $\frac{1}{2}$ " (38mm) stoma	81338	81438

One-Piece 11" Drainable Pouch with Filter

Box of 10

HCPCS A4424

- MicroSkin® Adhesive Barrier
- GORE-TEX® Charcoal Filter
- Thin washer

CLEAR



81300F

OPAQUE



81400F

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	81322F	81422F
For 1" (25mm) stoma	81325F	81425F
For 1 $\frac{1}{8}$ " (29mm) stoma	81329F	81429F
For 1 $\frac{1}{4}$ " (32mm) stoma	81332F	81432F
For 1 $\frac{3}{8}$ " (35mm) stoma	81335F	81435F
For 1 $\frac{1}{2}$ " (38mm) stoma	81338F	81438F

GORE-TEX® is a registered trademark of W. L. Gore & Associates



Colostomy One-Piece Pouching Systems

One-Piece 11" Press 'n Seal Drainable Pouch

Box of 10
HCPCS A4388

- MicroSkin® Adhesive Barrier
- Thin washer
- Press 'n Seal Closure

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	81322V
For 1" (25mm) stoma	81325V
For 1 $\frac{1}{8}$ " (29mm) stoma	81329V
For 1 $\frac{1}{4}$ " (32mm) stoma	81332V
For 1 $\frac{3}{8}$ " (35mm) stoma	81335V
For 1 $\frac{1}{2}$ " (38mm) stoma	81338V



81300V



**Press 'n Seal
Closure**
(Similar to Velcro®)

Simply fold up the
pouch 5 times and
press the locking flap
for a secure closure.

One-Piece 11" Press 'n Seal Drainable Pouch with Filter

Box of 10
HCPCS A4424

- MicroSkin® Adhesive Barrier
- Gore-tex® Charcoal Filter
- Thin washer
- Press 'n Seal Closure

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	81422V
For 1" (25mm) stoma	81425V
For 1 $\frac{1}{8}$ " (29mm) stoma	81429V
For 1 $\frac{1}{4}$ " (32mm) stoma	81432V
For 1 $\frac{3}{8}$ " (35mm) stoma	81435V
For 1 $\frac{1}{2}$ " (38mm) stoma	81438V



81400V



**Press 'n Seal
Closure**
(Similar to Velcro®)

Simply fold up the
pouch 5 times and
press the locking flap
for a secure closure.

Colostomy One-Piece Pouching Systems

One-Piece 11" Drainable Pouch with Plain Barrier Box of 10 HCPCS A4388

- MicroSkin® Adhesive Barrier

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm) stoma

CLEAR



81100

OPAQUE



81120

One-Piece 11" Press 'n Seal Drainable Pouch with Plain Barrier

Box of 10
HCPCS A4388

- MicroSkin® Adhesive Barrier
- Press 'n Seal Closure

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm) stoma



81100V



**Press 'n Seal
Closure**
(Similar to Velcro®)

Simply fold up the
pouch 5 times and
press the locking flap
for a secure closure.

One-Piece 9" Drainable Pouch with Plain Barrier Box of 10 HCPCS A4388

- MicroSkin® Adhesive Barrier

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

CLEAR



41100

OPAQUE



41120

Colostomy One-Piece Pouching Systems

One-Piece 9" Drainable Pouch

Box of 10
HCPCS A4388

- MicroSkin® Adhesive Barrier
- Thin washer

CLEAR



41300

OPAQUE



41400

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	41322	41422
For 1" (25mm) stoma	41325	41425
For 1 $\frac{1}{8}$ " (29mm) stoma	41329	41429
For 1 $\frac{1}{4}$ " (32mm) stoma	41332	41432
For 1 $\frac{3}{8}$ " (35mm) stoma	41335	41435
For 1 $\frac{1}{2}$ " (38mm) stoma	41338	41438

One-Piece 8" Closed-End Pouch

Box of 15
HCPCS A4416

- MicroSkin® Adhesive Barrier
- Charcoal Filter
- Thin Washer

OPAQUE



85401*

OPAQUE



85400

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

* This product is available
in pre-cut sizes

- Plain barrier with
no washer



Ben Moon was diagnosed with colo-rectal cancer at the age of 29. Now with a permanent colostomy, he is back to doing all of the things he loves. He makes a living as an adventure photographer (his Web site is moonfoto.com) and he enjoys surfing, climbing, and yoga. All of these sports demand a lot from an ostomy appliance. Ben uses Cymed's MicroSkin® pouches. He says, "No other product on the market even remotely works in the water the way MicroSkin® does."

Colostomy Two-Piece Pouching Systems

Two-Piece System Barriers

Box of 5

HCPCS A4410

- MicroSkin® Adhesive Barrier
- Universal flange works with all Cymed two-piece pouches (except 68100)

Plain Barrier



58100

Thin Washer



58000

Thick Washer



78000

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma

58022

78022

For 1" (25mm) stoma

58025

78025

For 1 $\frac{1}{8}$ " (29mm) stoma

58029

78029

For 1 $\frac{1}{4}$ " (32mm) stoma

58032

78032

For 1 $\frac{3}{8}$ " (35mm) stoma

58035

78035

For 1 $\frac{1}{2}$ " (38mm) stoma

58038

78038

Two-Piece Drainable Pouch Options

Box of 10

HCPCS A5063

- For use with all regular Cymed two-piece Barriers

11" pouch	9" pouch
 51445	 41445
51345	41345
51445	41445

CLEAR

OPAQUE


- For use with all regular Cymed two-piece Barriers
- Press 'n Seal Closure



Press 'n Seal Closure

(Similar to Velco®)

Simply fold up the pouch 5 times and press the locking flap for a secure closure.

11" pouch	11" pouch
	
78445V no filter	51445V with GORE-TEX® filter

Colostomy Two-Piece Pouching Systems

Two-Piece Large-Size Drainable System

- Barrier cut-to-fit for stomas up to 2 1/2" (64 mm)
- MicroSkin[®] Adhesive Barrier



68100
Box of 5
HCPCS A4415

CLEAR



68364
Box of 10
HCPCS A5063

OPAQUE



68464
Box of 10
HCPCS A5063

* Large Barrier washer now available separately. (See pg. 15 for more details)

Two-Piece Closed-End Pouch Options

- GORE-TEX[®] Charcoal Filter
- For use with all Cymed regular two-piece Barriers
- Opaque



8" pouch
55645
Box of 15
HCPCS A4419



6" pouch
35645
Box of 10
HCPCS A4419



Stoma Cap
25645
Box of 15
HCPCS A5055
Includes absorbent liner

Two-Piece System Irrigation Sleeve

- For use with all regular Cymed two-piece Barriers

59345

Box of 10
HCPCS A4397



Ileostomy One-Piece Pouching Systems

One-Piece 11" Drainable Pouch

Box of 10

HCPCS A4388

- MicroSkin® Adhesive Barrier
- Thick Washer

CLEAR



78300

OPAQUE



78400

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	78322	78422
For 1" (25mm) stoma	78325	78425
For 1 $\frac{1}{8}$ " (29mm) stoma	78329	78429
For 1 $\frac{1}{4}$ " (32mm) stoma	78332	78432
For 1 $\frac{3}{8}$ " (35mm) stoma	78335	78435
For 1 $\frac{1}{2}$ " (38mm) stoma	78338	78438

One-Piece 9" Drainable Pouch

Box of 10

HCPCS A4388

- MicroSkin® Adhesive Barrier
- Thick Washer

CLEAR



48300

OPAQUE



48400

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	48322	48422
For 1" (25mm) stoma	48325	48425
For 1 $\frac{1}{8}$ " (29mm) stoma	48329	48429
For 1 $\frac{1}{4}$ " (32mm) stoma	48332	48432
For 1 $\frac{3}{8}$ " (35mm) stoma	48335	48435
For 1 $\frac{1}{2}$ " (38mm) stoma	48338	48438



Ileostomy One-Piece Pouching Systems

One-Piece 11" Drainable Pouch

Box of 10
HCPCS A4388

- MicroSkin® Adhesive Barrier
- Thick washer
- Press 'n Seal Closure

Cut-to-fit for Stomas
up to 1 $\frac{3}{4}$ " (45mm)

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	78422V
For 1" (25mm) stoma	78425V
For 1 $\frac{1}{8}$ " (29mm) stoma	78429V
For 1 $\frac{1}{4}$ " (32mm) stoma	78432V
For 1 $\frac{3}{8}$ " (35mm) stoma	78435V
For 1 $\frac{1}{2}$ " (38mm) stoma	78438V



78400V



**Press 'n Seal
Closure**
(Similar to Velcro®)

Simply fold up the
pouch 5 times and
press the locking flap
for a secure closure.

The Original ILE-SORB®

HCPCS A4422

Soluble absorbent granule packets for ileostomy effluent.

The Original ILE-SORB® granules absorb the ileostomy fluid and evenly distribute the weight.

- Increases security
- Keeps effluent away from barrier —can increase wear time
- Makes emptying more manageable
- Plastic film packet enclosing the granules dissolves inside pouch



87210 90 packets
87230 135 packets



John Dermengian, an Ironman triathlete, uses MicroSkin® Ostomy Pouches for his ileostomy. John finished the grueling Ironman Triathlon in Hawaii in October of 2003. He swam 2.4 miles, rode 112 miles on his bicycle and then ran 26.2 miles. His MicroSkin® ostomy pouch passed the test!

Ileostomy Two-Piece Pouching Systems

Two-Piece System Barriers with Thick Washer

Box of 5
HCPCS A4410

- MicroSkin® Adhesive Barrier
- Universal flange works with all Cymed two-piece pouches (except 68100)



Cut-to-fit for Stomas up to 1 $\frac{3}{4}$ " (45mm) 78000

Pre-cut sizes:

For $\frac{7}{8}$ " (22mm) stoma	78022
For 1" (25mm) stoma	78025
For 1 $\frac{1}{8}$ " (29mm) stoma	78029
For 1 $\frac{1}{4}$ " (32mm) stoma	78032
For 1 $\frac{3}{8}$ " (35mm) stoma	78035
For 1 $\frac{1}{2}$ " (38mm) stoma	78038

Two-Piece 11" Drainable Pouch Options

Box of 10
HCPCS A5063

11" pouch	11" pouch	11" pouch	9" pouch
78445V Press 'n Seal Closure	51445V Press 'n Seal Closure with GORE-TEX® filter	78445	41445

Two-Piece Large-Size Drainable System

- Barrier cut-to-fit for Stomas up to 2 $\frac{1}{2}$ " (64 mm)
- MicroSkin® Adhesive Barrier



68100
Box of 5
HCPCS A4415



68464
Box of 10
HCPCS A5063

- * Large Barrier washer now available separately. (See pg. 15 for more details)

The Cymed Hydrocolloid Washer

Canister of 30

HCPCS A4404

Washers provide extra protection and support to the skin surrounding the stoma without sacrificing comfort. Naturally extracted from citrus peels, Cymed's washers are a specially formulated pectin-based hydrocolloid. Washers can provide some absorption and longer wear time for wounds, as well as providing some minor convexity for recessed stomas.



Cut-to-fit for Stomas up to 1½" (38mm)

Pre-cut sizes:

For 7⁄8" (22mm) stoma

For 1" (25mm) stoma

For 1 1⁄8" (29mm) stoma

For 1 1⁄4" (32mm) stoma

For 1 3⁄8" (35mm) stoma

For 1 1⁄2" (38mm) stoma

Large Sizes:

Cut-to-fit for Stomas up to 2½" (64mm)

THIN	THICK
89410	78910
89422	78922
89425	78925
89429	78929
89432	78932
89435	78935
89438	78938
99410	98910

The Cymed Seal

HCPCS A4385

Moldable, stretchable washer fits any stoma. Fills irregularities, and provides a waterproof seal and skin protection.

CS 1000 10 seals per package

CS 1020 20 seals per package

CS 1030 30 seals per package



Micro Skin® Applicator

Optional for use applying MicroSkin® Adhesive Barrier. The mirror reflects the stoma and allows you to line up the stoma in the mirror to place the pouch accurately.

89180



Cavilon™

Cavilon™ by 3M, is an alcohol-free wipe that dries quickly to form a breathable coating on the skin. Cavilon™ will help protect the skin and may also aid the MicroSkin® to adhere to your body and achieve greater wear times.

3M-3344 Box of 25 individual wipes

3M-3346 28 mL Bottle

HCPCS A5120

HCPCS A4369




Pediatric Pouching Systems

One-Piece 7" Fistula Pouch

Box of 10

HCPCS A4391

- MicroSkin® Adhesive Barrier
- Anti-Reflux Design
- Supplied with Two Drainage Adapters

Plain Barrier	Thin Washer
	
86400	86400W
Cut-to-fit for Stomas up to 1 3/4" (45mm)	Cut-to-fit for Stomas up to 1 3/4" (45mm)

One-Piece 7" Drainable Pouch Options

Box of 10

HCPCS A4424

- MicroSkin® Adhesive Barrier
- GORE-TEX® Charcoal Filter

Plain Barrier	Thin washer	Thick washer
		
81530	81530W	78530
Cut-to-fit for Stomas up to 1 3/4" (45mm)	Cut-to-fit for Stomas up to 1 3/4" (45mm)	Cut-to-fit for Stomas up to 1 3/4" (45mm)
81520	81520W	78520
81530	81530W	78530



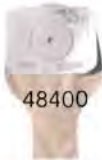
CLEAR
OPAQUE

One-Piece 9" Drainable Pouch

Box of 10

HCPCS A4388

- MicroSkin® Adhesive Barrier
- Three Barrier Options

Plain Barrier	Thin washer	Thick washer
		
41100	41400	48400
41100	41300	48300
41120	41400	48400

Cut-to-fit **CLEAR**
Cut-to-fit **OPAQUE**

Pediatric Pouching Systems

Two-Piece Barriers

Box of 5

HCPCS A4410

- MicroSkin® Adhesive Barrier with Universal Flange
- Three Barrier Options

Cut-to-fit

Plain Barrier	Thin washer	Thick washer
		
58100	58000	78000

For Pre-cut sizes refer to page 10

Two-Piece Pouch Options

- For use with all regular Cymed two-piece barriers

Two-Piece 6" closed end pouch	35645	Box of 10	HCPCS A4419	(*details on page 11)
Two-Piece 9" drainable pouch	41345	Box of 10	HCPCS A5063	(*details on page 10)
Two-Piece 9" drainable pouch	41445	Box of 10	HCPCS A5063	(*details on page 10)
Two-Piece 8" closed end pouch	55645	Box of 15	HCPCS A4419	(*details on page 11)



MicroSkin® Hints and Tips

Creating the best bond between your skin and MicroSkin®

In most cases, **MicroSkin adheres best on clean, dry skin.** Wash with plain water and dry the skin thoroughly. Avoid application in a steamy bathroom or if you are sweating.

What if the MicroSkin® doesn't stick well?

- If you must use soap around your stoma, use one with minimal moisturizers such as Ivory.
- Skin preps and powders diminish adhesion, and are not recommended to use with any MicroSkin product. Additionally, the use of any product between your skin and the MicroSkin can compromise the breathability of the MicroSkin (see pg. 3 for more information on Breathability).
- Washers (for more details on washers, see pg. 3) create a flexible, secure seal. Ensure washer size: washer should touch all the way around stoma without squeezing, leaving no visible gap.
- Skin irritation or breakdown a common reason for a pouch to not stay on. Moisture from red or broken skin can float the MicroSkin off. Care of skin irritation should be guided by your ostomy nurse or health care professional. You may also consider using 3M Cavilon™ (see pg. 15).

If there is leakage . . . the solution is identifying the root cause and to remove it. This may take some investigative work, then practice implementing a correction, but it's worth it. Until a solution is found, skin protection is very important (consider using 3M Cavilon™, see pg 15).

Possible causes and solutions for leakage are:

- Leaving a small gap between the washer and the stoma does not allow a seal to be formed and can create a "channel" for liquid to leak through. Solution: when putting on a pouch, ensure the washer is touching the stoma all around (use a CTF size if necessary), and a good seal is more likely.
- If no washer is used, ensure you press the MicroSkin firmly down on the skin when putting it on for best adhesion and to prevent tiny openings for liquid to seep through. If that doesn't work, consider Skin Tac—this is a wipe-on adhesive. It creates a tacky surface for the washer and MicroSkin to cling to.
- If irregular skin contour causes leakage, try the **Cymed Seal**. It is a moldable, stretchable hydrocolloid ring that conforms to and fills-in any irregular shape in the skin. If the washer erodes or melts too quickly, an initial seal was not created. Try using an additional washer (see page 15).
- If you've tried these steps and you still have leakage, call us.

Will Medicare pay for these pouches?

Yes! All Cymed pouches receive Medicare reimbursement. Several suppliers take assignment on Cymed products. Call 800 582-0707 for details, or see the supplier list which comes with our sample packs.



Basic Steps for MicroSkin® Application

MicroSkin® is easy to apply! The MicroSkin® barrier comes with protective paper on BOTH sides. The side strips allow you to handle the barrier without touching the adhesive. Hold the pouch so that you can look at the writing.



- 1 Fold down the top of each side strip creating a tab for later use. Remove the circular paper in the center if your pouch has a washer.



- 2 Remove the printed square cover paper from the MicroSkin® barrier.



- 3 Holding the vertical side strips, apply the pouch over the stoma.



- 4 Remove the strips of paper from the sidebars by pulling the tabs at the top of each strip, and press down the edges of the barrier.



- 5 Remove the paper frame from the non-adhesive side of the barrier. Feel the softness and flexibility of MicroSkin®!



CYMED

life without limitations

www.cymedostomy.com

8400 Carbide Ct, Ste B
Sacramento, CA 95828

Call or email for free samples

800 582-0707

samples@cymedostomy.com



Find **CYMED** on:



You **Tube**

